Dermatix™ Silikon Gel zur Narbenbehandlung

Was ist Dermatix?

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Silicone gel for scar reduction

What is Dermatix gel?

Dermatix is a Topical Silicone Gel that is transparent, self-drying and maintains the skin's moisture balance, while aiding the management of scarred skin surfaces resulting from surgery, burns and other injuries. Dermatix has shown to flatten, soften and smooth scars, relieve the itching and discomfort of scars as well as reduce the discoloration associated with scars.

Dermatix gel is easily applied to all areas of the skin, including the face, joints and flexures, and dries rapidly to form an invisible "sheet".

Dermatix was developed using the same basic long chain polymers as topical silicone gel sheeting, but it is available in a tube. It contains polysiloxane, silicon dioxide.

The gel is supplied in tubes of 15 g.

What is Dermatix gel used for?

Dermatix is used for the management of scars after the wound has healed and the skin surface is intact, and in the prevention of scar development in the form of keloids and hypertrophic scars resulting from general surgical procedures, trauma wounds and burns.

How should Dermatix gel be used?

- Dermatix must not be used on broken skin.
- Be careful if applying Dermatix around your eyelids to avoid contact with the eyes.
- The gel should be applied in a very thin layer to the affected area twice daily, following the instructions below:
 - Clean the area to be treated with mild soap and water, pat dry.
 - Gently spread a **very thin coat** over the affected site.
 - Allow to air-dry for 4-5 minutes before covering with clothing or make-up. **(Please be aware that Dermatix may stain clothing if it is not completely dry).**
 - If Dermatix has not dried within 4-5 minutes you have used too much; remove the excess and allow the drying process to continue.
 - Cosmetics can be applied over Dermatix following complete drying.
- Dermatix can also be used on children.

The recommended minimum duration of use is 2 months. Further benefits may result from extended use.

When should Dermatix gel not be used?

- Dermatix should not be applied to open or fresh wounds.
- Dermatix should not be placed in contact with mucous membranes, or applied too close to the eyes.
- Dermatix should not be applied over antibiotic skin preparations or other skin treatments and/or products.
- Dermatix should be applied to clean, dry skin only.

Are there any possible unwanted effects of Dermatix gel?

Rarely, the gel may cause redness, pain or irritation to the skin. If you are concerned about these or any other unwanted effects consult your pharmacist or doctor.

How should Dermatix be stored?

- Keep out of reach of children.
- Keep gel away from the heat.
- Store below 25 °C.
- Do not use after the expiry date (EXP) printed on the pack.

Further information

You can get more information about Dermatix from your pharmacist or doctor.

Dermatix is manufactured by: Advanced Bio-Technologies Inc.
Silverdale, WA 98383, USA

The representative in Europe is: ICN Pharmaceuticals Holland BV
2723 RM Zoetermeer, Holland

Date of preparation

January 2003